


CONSERVE PPE

Setup

View the PPE videos at med.emory.edu/PPE



After gown and glove removal...

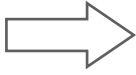

1 **Sanitize hands.**

2 **Put on fresh gloves.**

3 **Place wipe on table.**

4 **Remove eyewear and place on wipe.**

5 **Sanitize gloves.**

6 **Remove procedure mask, lower strap and then upper. Store mask.**

7 **Sanitize gloves.**

Face Shield

8 **Wipe front and back of shield.**

9 **Wipe elastic band.**

10 **Wipe foam band.**

11 **Wipe table.**

12 **Place shield upside down to dry.**

13 **Sanitize gloves.**

14 **Remove gloves.**

15 **Wash hands with soap and water.**

Goggles

8 **Wipe front and back of lens.**

9 **Wipe both ear pieces.**

10 **Wipe table.**

11 **Place goggles on clean table to dry.**

12 **Sanitize gloves.**

13 **Remove gloves.**

14 **Wash hands with soap and water.**